


Steps to Install the Chair Occupancy Sensor

1. Check the sensor for any visible defects or damage. If damage is visible the sensor pad should be replaced.
2. Verify that the pad is attached to the port on the white control box (shown below).

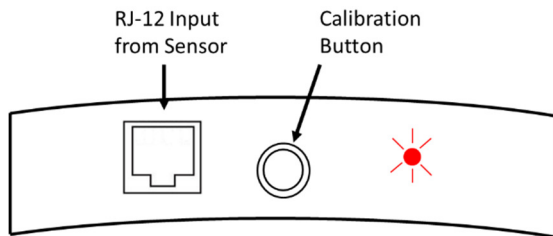

Locate the Chair Occupancy Sensor Mat on top of the chair seat, blue side up.

3. To improve patient comfort, a towel or thin blanket can be placed over the Sensor Mat.
4. The Sensor Mat should be oriented such that the cable and control box are positioned to the rear of the chair.
5. If the control box is new, pull out the plastic battery tab located on the underside of the unit. The tab is not needed for use and can be discarded.
6. Secure the control box to the chair by either (a) adhering it to the side of the chair or (b) tucking the control box between the chair padding and frame.
7. Verify that all cables have been secured to eliminate tripping hazard, and located in a place that prevents cable pinching.
8. Installation of the Chair Occupancy Sensor is now complete.

Note for Sensor Signal Delay Time:

The sensor may have been preprogrammed with a signal delay, typically between 3-10 seconds. Please refer to the SmartBox Part Number to determine the preset delay time.

Cleaning the Sensor Pad

1. Wipe down with moistened cloth using only soap and water, and wipe dry. Avoid using any cleaning agent containing bleach (sodium hydroxide) as an ingredient. **DO NOT MACHINE WASH!**
2. Carefully wipe down and clean the cable, checking for any damage that may have occurred.
3. **DO NOT** immerse the pad in any liquid.
4. After cleaning the sensor, repeat all of the installation steps when replacing the sensor.

Storage

1. Store the sensors flat and **DO NOT** store the sensor folded over.
2. The sensors are intended for normal indoor use in relatively low humidity environments.

Care and Use

1. Never pull on or jerk on the cable. Doing so may damage the connector or the mat.
2. Verify that the pad is lying flat on the chair seat and that it is not bent or folded over.
3. Replace the batteries every two years by removing the top cover of the white control box. The device takes (2) AAA batteries.